

How to USE AN AED (Automated External Defibrillator)

All automated external defibrillators come with easy-to-follow directions, but using one is even easier if you remember to D.R.A.W.!

> **DRY** the casualty's surroundings before you administer a shock to avoid injury to yourself.




R

REMOVE obstructions, like clothing, jewelry or coarse body hair from the casualty's chest before applying the electrode pads included with your AED.





APPLY the electrode pads diagonally across the casualty's chest as shown.

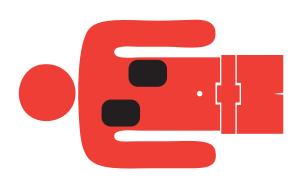



WARN bystanders and first responders before administering a shock so that they can keep their distance from the casualty.

For more information about AED use, sign up for a CPR and Emergency First Aid training course online, over the phone, or at your local St. John Ambulance branch. Or visit sja.ca

*Source: Automated External Defibrillators (AEDs). Government of Canada, 2016.